

MAINTENANCE

OF MC2 AND MC3 CARBON MICROMOTORS.



Refer to the user manual for the product for the complete cleaning and sterilisation maintenance procedures. This medical device is supplied in "non sterile" condition. Please ensure that this procedure is followed before the device is used.

1 PREPARATION



A. DISCONNECT THE MOTOR FROM THE CABLE

B. REMOVE THE SLEEVE


2 CLEANING



A. EXTERNAL CLEANING

- External cleaning with Spraynet

3 DISINFECTION

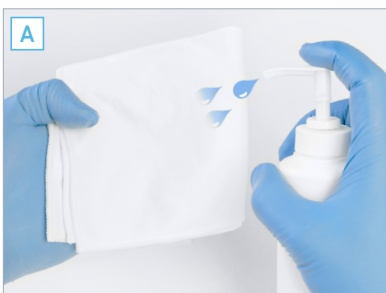

A. DISINFECTION USING SURFACE DISINFECTANT
(follow the manufacturer's instructions for use)


- ⊘ Do not immerse the instrument in a disinfectant bath

4 STERILISATION



A. PLACE THE SLEEVE FOR THE MOTOR IN A STERILISATION BAG

B. STERILISE IN ACCORDANCE WITH THE MANUFACTURER'S INSTRUCTIONS

- Instruments bearing the  symbol can be sterilised in an autoclave

MAINTENANCE ACCESSORIES



SPRAYNET
500 ml cleaning spray
Pack of 6
REF. 1600036-006



LUBRIFLUID
500 ml lubricant
Pack of 6
REF. 1600064-006



AQUACARE
500 ml cleaning spray
Pack of 6
REF. 1600617-006



LUBRICARE 2
Maintenance system
REF. 1601156-001



LUBRIMED
Pack of 6 cartridges
REF. 1600037-006



BIEN-AIR CLEANING WIRE
Pack of 10
REF. 1000001-010