www.bienair.com

Other addresses available at

dental@bienair.com

Tel. +41 (0)32 344 64 64 Fax +41 (0)32 344 64 91
Länggasse 60 Case postale 2500 Biènnne 6 Switzerland
Bien-Air Dental SA

Bien Air[®]
Dental

Bien Air[®]
Dental

Micromotors MX-i MX-i LED 3rd Gen MX-i LED

ENG	Instruction
FRA	Mode d'emploi
DEU	Anleitung
ESP	Instrucciones de uso
ITA	Istruzioni
PRT	Instruções
RUS	Инструкция по эксплуатации

REF 2100245-0006/2019.03

CE
0120

a Set supplied

MX-i

REF 1600825-001

MX-i LED 3rd Gen


REF 1601008-001

MX-i LED

REF 1600755-001

b Associated products

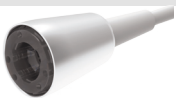REF 1600606-001
(For MX-i and
MX-i LED)

REF 1601069-001
(For MX-i and
MX-i LED)

REF 1601009-001
(Only for MX-i LED 3rd Gen)

REF 1300967-010
(For all MX-i micromotors)

Assembly

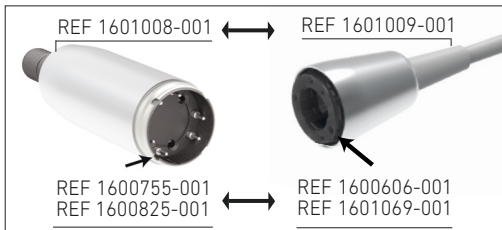
fig. 1

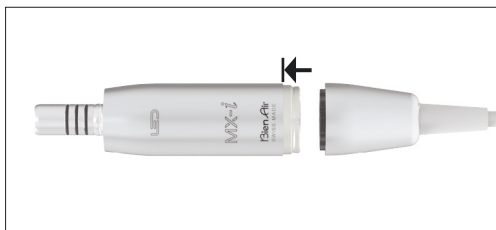
fig. 2A

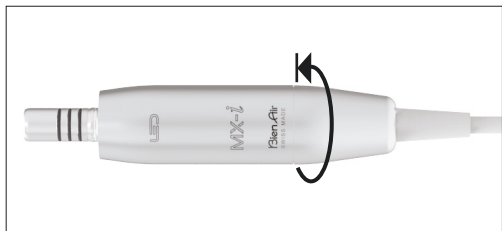
fig. 2B

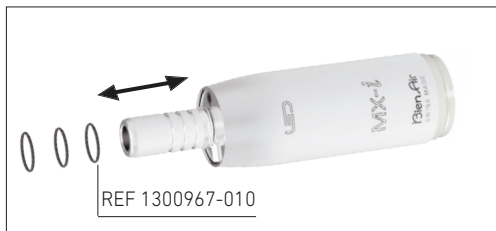
fig. 3

Cleaning Instructions

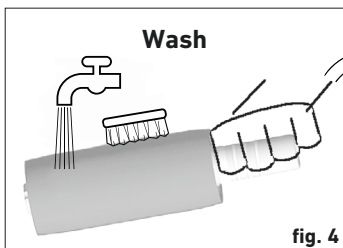
fig. 4

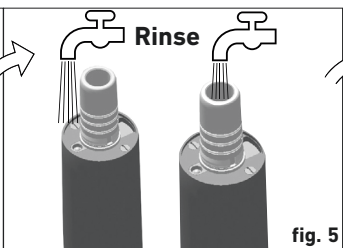
fig. 5

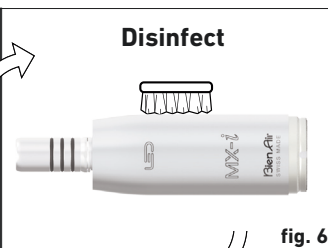
fig. 6

fig. 9

fig. 8

Wash in a Thermal Disinfector

fig. 7

This product may be covered by one or more of the following patents:

EP Europe: 745358 / 688539 / 948294 / 1145688 / 1563800 / 1563801 / 1675523 / 1753360 DE Germany: 29616023.7

DK Denmark: 9600315 FR France: 2722972 CH Switzerland: 693922 CN China: 100528099 / 100522100 / 100522099 / 100553584

JP Japan: 3892485 / 4298933 / 7000419 US United-States: 5453008 / 6033220 / 6319003 / 7214060 / 7448870

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English

In these instructions, "Device" corresponds to the micromotors described in the heading "Type".

Type

REF	Legend
1601008-001	MX-i LED 3 rd Gen micromotor with light, sterilisable, machine washable
1601009-001	Cable for MX-i LED 3 rd Gen micromotor, sterilisable, machine washable
1600755-001	MX-i LED micromotor with light, sterilisable, machine washable
1600825-001	MX-i micromotor without light, sterilisable, long nose, machine washable
1600606-001	Cable for MX-i LED and MX-i micromotors, sterilisable, non machine washable

1601069-001 Cable for MX-i LED and MX-i micromotors, sterilisable, machine washable

1300967-010 O-Ring for MX-i LED and MX-i micromotors

Important

These micromotors must only be used with Bien-Air Dental cable and electronic controls. Otherwise, performance or safety can not be guaranteed.

Intended use

Product intended for professional use only. Use it in general dentistry, implantology, oral and maxillofacial surgery. The device is not designed for use in an explosive atmosphere (anaesthetic gas).

Technical data and assembly

Classification

Class IIa in accordance with European Directive 93/42/EEC concerning medical devices.

Coupling

Nose in accordance with ISO 3964, without internal spray.

Dimensions MX-i

Ø 23.2 x 100.4 mm, incl. the nose.

Ø 23.2 x 68.7 mm, without the nose.

Dimensions MX-i LED 3rd Gen

Ø 23.2 x 83.8 mm, incl. the nose.

Ø 23.2 x 61.8 mm, without the nose.

Dimensions MX-i LED

Ø 23.2 x 90.9 mm, incl. the nose.

Ø 23.2 x 68.7 mm, without the nose.

Weight (without cable)

115 g (MX-i and MX-i LED).

110 g (MX-i LED 3rd Gen).

Important

Actual performances and safety are dependent on the type of electronic supply used. To reach optimal performance, only use electronic provided by Bien-Air Dental.

Important

Never connect an instrument on a running micromotor.

Junction

Only with a REF 1600606-001, 1601069-001 or 1601009-001 power cable depending of the motor. Check that the rear of the motor and the cable connector are clean and dry. Position the motor as shown in **fig. 1**. The locator must be in front of the hole. Rotate it to find the exact position and push in connector **fig. 2A**. Tighten the nut to the stop **fig. 2B**.

Changing the seals REF 1300967-010

Refer to fig. 3.

Standards (all MX-i micromotors)

Complies with the ISO 11498 (dental handpieces – Low voltage electric dental motors). The device must be classified as type B applied part. The corresponding terminology is defined in section 3.132 of the IEC 60601-1 (Medical electrical equipment – Part 1: General requirements for basic safety and key performances).

Standards (MX-i LED 3rd Gen only)

Complies with the ISO 14457 (Dentistry - Handpieces and motors) and IEC 80601-2-60 (Medical electrical equipment).

Rotation speed

From 100 to 40,000 rpm $\pm 5\%$, clockwise and counterclockwise.

Mechanical power

Max 140 W $\pm 5\%$ at 25,000 rpm.

Torque

Chiropro L : 6.3 Ncm.

iChiropro : 5.0 Ncm.

Chiropro : 5.0 Ncm.

Chiropro Plus : 5.5 Ncm.

Supply voltage / Max admissible current

32VDC / 10A peak

Noise level

Less than 60 dBA at 45 cm.

Luminous intensity MX-i LED / MX-i LED 3rd Gen

Greater than 12 klux with CA 20:1 L REF 1600598-001.

Disposal

This device must be recycled. Electrical and electronic equipment may contain dangerous substances which constitute health and environmental hazards.

The user must return the device to its dealer or establish direct contact with an approved body for treatment and recovery of this type of equipment (European Directive 2002/96/ EC).

Maintenance

Important

- The instrument is delivered "non sterile".
- Before using for the first time and within a maximum of 30 minutes after each treatment, clean, disinfect the attachment, then sterilise. Observing this procedure eliminates any blood, saliva or saline solution residues.
- Do not immerse in an ultrasonic bath.
- Only use original Bien-Air Dental maintenance products and parts or those recommended by Bien-Air

Dental. Using other products or parts may cause operational failure and/or void the guarantee.

In the event of prolonged nonuse, the instrument must be stored in a dry environment. Clean and sterilise the instrument before reuse.

Precautions of use

The standard precautions in particular wearing individual protection equipment (gloves, glasses...), must be complied with by healthcare personnel working with contaminated or potentially contaminated medical instruments. Pointed and sharp instruments should be handled with great caution. Check that the steriliser and the water that is used are clean. After each sterilisation cycle, remove the device from the sterilising apparatus immediately, in order to reduce the risk of corrosion. We recommend that the motor is cleaned and sterilised as described below before the initial first use and subsequently after each treatment.

1 Manual cleaning

Washing

The external surface of the motor must be cleaned to remove impurities as shown in **fig. 4**.

- Hold the motor by the nose under running water (< 38°C – 149°F)
- With the aid of a soft bristled brush, clean the external surface of the motor.

Rinsing

Rinse the exterior and interior of the motor nose under running water for 10 seconds. (< 38°C – 149°F) **fig. 5**.

Disinfection

- Carefully rub the external surfaces of the motor, for approximately one minute, with a soft bristle brush impregnated with a detergent or disinfectant solution **fig. 6**.
- Rinse the exterior of the motor under running water (< 38°C – 149°F).

Suitable detergents

- Detergent or detergent-disinfectant (pH 6- 9.5) recommended for cleaning-disinfection of dental or surgi-

cal instruments.

- Quaternary ammonium- and/or enzyme-based surfactants.
- Do not use solutions that are corrosive or contain chlorine, acetone aldehydes or bleaches.
- Do not soak in physiological liquid (NaCl).

2 Automatic cleaning-disinfection

 Note:

only for instruments engraved with

Cleaner - disinfectant

Use a validated cleaner-disinfectant with the adequate detergent.

Washing cycle

Select the specified washing cycle for these instruments which should be compatible with the detergent manufacturer's instructions.

Recommended temperatures:

Pre-wash

Water: cold to maximum 35°C (95°F)
Duration 2 mins.

Wash

Water: 50 to 65°C (122 to 149°F)
Duration 5 mins.

Rinsing

Water: cold to maximum 35°C (95°F)
Duration 2 mins.

Thermal disinfection

Water: 80 to 97°C (176 to 206.6°F)
Duration 5 mins.

Drying

Air: 65 to 75°C (149 to 167°F)
Duration 25 mins.

 Note: Never rinse the devices to cool them.

3 Sterilisation

Do not use a sterilisation procedure other than the one described below.

Procedure:

Fractionated pre-vacuum steam sterilisation, Class B cycle acc. to EN13060. The procedure has been validated according to ISO 17665. All Bien-Air Dental electric micromotors are sterilisable in an autoclave up to 135°C (275°F). Duration: 3 or 18 min., depending to the national requirements in force.

Important

The quality of the sterilisation depends very much on the cleanliness of the device. Only perfectly clean devices may be sterilised **fig. 8**.

The instrument will function in excess of 500 sterilisations.

4 Maintenance

Important

Do not spray any lubricant or cleaning solution into the motor fig. 9!

Servicing

Never disassemble the device. For all modification and repair, we recommend that you contact your sales representative or an official Bien-Air Dental repair center.

Bien-Air Dental recommends the user to have its dynamic instruments checked or inspected at least once a year.

Transportation and storage conditions

Temperature between -40°C (-40°F) and 70°C (158°F), relative humidity between 10% and 100%, atmospheric pressure 50 kPa to 106 kPa (7.3 to 15.3 psi).

Other precautions for use

The device must be used by a qualified professional in compliance with the current legal provisions concerning workplace safety, health and accident prevention measures, and these working instructions. In accordance with these requirements, the operators:

- must only use operating devices that are in perfect working order; in the event of irregular functioning, excessive vibration, abnormal heating or other signs indicating malfunction of the device, the work must be stopped immediately; in

this case, contact a repair centre that is approved by Bien-Air Dental;

- must ensure that the device is used only for the purpose for which it is intended, must protect themselves, their patients and third parties from any danger, and must avoid contamination through the use of the product.

Rest the device on a suitable support to avoid risks of infection for yourself, the patient or third parties.

To avoid the tubing and connectors from blocking, the water should be filtered and purified to avoid impurities and build-up of high levels of calcium deposits.

Guarantee

Terms of guarantee

Bien-Air Dental grants the operator a guarantee covering all functional defects, material or production faults.

The device is covered by this guarantee for 36 months from the date of invoicing.

In the event of justified claim, Bien-Air Dental or its authorised representative will fulfil the company's obligations under this guarantee by repairing or replacing the product free of charge. Any other claims, of whatever nature, in particular in the form of a claim for damages and interest, are excluded.

Bien-Air Dental shall not be held responsible for damage or injury and the consequences thereof, resulting from:

- excessive wear and tear
- improper use
- non-observance of the instructions for installation, operation and maintenance
- unusual chemical, electrical or electrolytic influences
- poor connections, whether of the air, water or electricity supply.

The guarantee does not cover flexible "fibre optic" type conductors, or any parts made of synthetic materials.

The guarantee shall become null and void if the damage and its consequences are due to improper manipu-

lation of the product, or modifications to the product carried out by persons not authorised by Bien-Air Dental.

Claims under the terms of the guarantee will be considered only on presentation, together with the product, of the invoice or the consignment note, on which the date of purchase, the product reference and the Serial No. should be clearly indicated.

a Set supplied

See cover page.

b Optional accessories

See cover page.

Cable compatibility

Cable	Micromotor	Console
1600606-001	1600755-001 1600825-001	1600724-001 1600613-001 1600784-001 1600994-001
1601069-001	1600755-001 1600825-001	1600995-001
1601009-001	1601008-001	1600724-001 1600613-001 1600784-001 1600994-001

Symbols

Manufacturer.

Attention.

Light.

Machine washable.

Sterilisable at the specified temperature.

Wear rubber gloves.

CE Marking with number of the notified body.

Electrical security.
Applied part type B.

Separate collection of electric and electronic equipment

Move in the direction indicated.

Move fully to the stop, in the direction indicated.

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