Bien-Air Dental SA Länggasse 60 Case postale 2500 Bienne 6, Switzerland Tel. +41 (0)32 344 64 64 Fax +41 (0)32 344 64 91 dental@bienair.com

Bien-Air France Sàrl 19-21, rue du 8 Mai 1945 94110 Arcueil France Tel. +33 (0)1 49 08 02 60 Fax +33 (0)1 46 64 86 68 ba-f@bienair.com

Bien-Air España S.A.U. Entença, 169 Bajos 08029 Barcelona, España Tel. (+34) 934 25 30 40 Fax (+34) 934 23 98 60 ba-e@bienair.com

Bien-Air USA, Inc. Medical Technologies 8861 Research Drive Irvine, CA 92618, USA Phone 1-800-433-BIEN Phone 949-477-6050 Fax 949-477-6051 ba-usa@bienair.com

Bien-Air UK Limited Arundel House Unit 1 - Ground Floor Amberley Court Whitworth Road Crawley, West Sussex RH11 7XL, England Tel. +44 (0)1293 550 200 Fax +44 (0)1293 520 481 ba-uk@bienair.com

Bien-Air on Internet: www.bienair.com Bien-Air

Deutschland GmbH Jechtinger Strasse 11 79111 Freiburg, Deutschland Tel. +49 (0)761 45 57 40 Fax +49 (0)761 47 47 28 ba-d@bienair.com

Bien-Air Italia s.r.l. Via Vaina 3 20122 Milano, Italia Tel. +39 (02) 58 32 12 51/52/54 Fax +39 (02) 58 32 12 53 ba-i@bienair.com


Bien-Air Asia Ltd. Nishi-Ikebukuro Daiichi-Seimei Bldg. 10F 2-40-12 Ikebukuro, Toshimaku Tokyo, 171-0014, Japan ビエン・エア・アジア株式会社(製造販売元) 〒171-0014 東京都豊島区池袋2-40-12 西池袋第一生命ビルディング10F Tel. +81 (3) 5954-7661 Fax +81 (3) 5954-7660

ba-asia@bienair.com

Beijing Bien-Air Medical Instrument Technology Service Co. Ltd. Room 907, The Exchange Beijing, No 118 Jian Guo Lu Yi, Chao Yang District, Beijing 100022, China 北京彼岸医疗器械 技术服务有限公司 北京市朝阳区建国路 乙118 号招商局中心 京汇大厦907室 电话+86 10 6567 0651 传真+86 10 6567 8047

ba-beijing@bienair.com



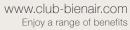







IMPLANTOLOGY SYSTEMS A TOUCH OF GENIUS



"THE PHYSIODISPENSERS THAT REALLY MEET MY EXPECTATIONS"

PROF. DR. DANIEL BUSER

3

Implantology is a demanding discipline, which Always at the forefront of innovation, Bien-Air is why Bien-Air leaves nothing to chance. now offers systems to rival all competitors. Created by practitioners for practitioners, our physiodispensers are highly efficient, extremely versatile and designed for a range of clinical applications.

Prof. Dr. Daniel Buser is the author and co-author of over 300 publications in national and international journals. He has also written and co-written many works including the GBR book (Guided Bone Regeneration in Implant Dentistry) Editions I (1993) and II (2008). Prof. Buser is president of the Swiss Implant foundation and a member of the Osteology Foundation. He is former president of the International Team for Implantology (ITI), the European Association for Osseointegration (EAO), the Swiss Society of Oral Surgery (SSIO) and the Swiss Society of Oral Surgery and Stomatology (SSOS). He has been awarded the André Schroeder research prize (1995) and the Brånemark Osseointegration Award (2013), amongst others.

BIEN-AIR REINVENTS IMPLANTOLOGY FOR GREATER EASE OF USE

Our unique, intuitive and comprehensive interfaces guarantee unrivalled performance and ease of use. The high torque of Bien-Air micromotors makes them extremely easy to work with, at both high and low speeds. Furthermore, our lifetime-lubricated ceramic ball bearings guarantee you exceptional service life and reliability. Our contra-angles feature a unique internal irrigation system which ensures optimal handling. You mentioned user-friendliness?

- Complete operating sequences for the main pre-programmed implant systems
- Customisable programs
- Simple, intuitive control interfaces
- The most reliable and efficient micromotor
- A contra-angle equipped with one of the smallest heads on the market
- LED lighting with constant intensity at all rotation speeds
- Natural light for a realistic view of the dentine, pulp or gingiva
- Ingenious, exclusive peristaltic pump.



5



USER MIND

Bien-Air has reinvented medical instrumentation by combining technological intelligence and the user experience. Ingenuity and pleasure go hand in hand with unrivalled ease of use.



MICRO-SERIES

More compact and lighter assemblies, for exceptional balance and handling.



CARE FREE

Thanks to its lifetime-lubricated ceramic ball bearings, your micromotor requires no lubrication. This makes it quicker to re-use and reduces maintenance costs.



QUALITY LED

Bien-Air is setting new standards in the field of LED lighting. Intensity adjustable from 0 to 38,000 lux, perfectly natural light, exceptional sterilisation resistance.



DUALOOK

The two glass fibre rods provide the best lighting on the market and are robust enough to withstand all conditions.



SMART LOGIC

The most high-precision control ever obtained. Torque, speed and bur position controlled to perfection, even at low speeds.



DATA CONTROL

The user can take charge of the technology thanks to unique data management. Preferences, settings and documents can be personalised, saved, printed, exported or sent.



SWISS MADE

Exceptional techniques and materials for products which are individually manufactured and tested in the heart of Watch Valley, in Switzerland.









ICHIROPRO A REVOLUTION FROM BIEN-AIR











The iChiropro is the implantology and surgery Available for free download from the Apple App system everyone's talking about. It is no exag- Store, the iChiropro application can be customgeration to say that this little wonder is a revoluised as desired. In addition, new innovative tionary product in terms of innovation, perfor- functions will regularly be added to further enmance and ease of use.

With iChiropro, operations are made significantly easier thanks to an intuitive and ergonomic iPad application.

hance the capability and performance levels of your physiodispenser.

iChiropro, the new device controlled from an iPad and equipped with some of the best performing instruments on the market, paves the way for a new generation of medical equipment. Its innovative solutions make practitioners' lives easier that's the Bien-Air promise.

Video, photo gallery, contacts, special offers and orders via www.ichiropro.com









The iChiropro is the only system enabling multi-implant operations. Optimise your operations by treating up to 8 implants simultaneously.

- Multiple users
- Simplified operations
- Multiple implants
- Plan multiple operations
- Pre-programmed operating sequences
- Customised sequences
- Implant barcode reader
- Integrated patient file
- Operation report
- Back-up, export and send data

The iChiropro is programmed for over 20 implant systems, including:

Nobel Biocare - Straumann - Thommen Medical - Camlog - Dentsply Implants - Bredent - Euroteknika - Bti - BioHorizons - MIS - Neoss - Dio - Camlog...

The above implant brands are trademarks registered by companies of the same name. The parameters contained in the implant manufacturers' programs are given for information only and Bien-Air cannot be held liable for them. The parameters may be subject to modification without notice.





Save a lot of time by combining ease of use Thanks to its barcode reader, the iChiropro and efficiency. The iChiropro integrates full op- enables you to instantly identify the implant erating sequences from the leading implant being used in order to ensure data accuracy manufacturers on the market and enables you and traceability. to plan and program multi-implant operations in advance of your upcoming appointments. All An operation report containing graphics and settings can be customised and saved according to individual user needs, from preparation of the implant site to insertion of the implant.

The operational parameters for the iChiropro implantology and surgery system are directly modifiable, to facilitate your operations: speed, torque, direction of rotation, irrigation flow rate, lighting intensity, data saving, etc.

tables can be attached to your patient file. You can consult, save, print, export or send it.

iChiropro - The smart way to your success.









CHIROPRO L AND CHIRO FICIENCY AND VERS
















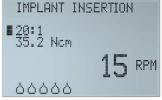



11


tems on the market. Their complete pre-pro- highly practical peristaltic pump, designed and grammed clinical sequences are displayed on patented by Bien-Air. Our irrigation lines incorthe screen. Chiropro L can also be custom- porate an insertion support ensuring very easy ised according to your needs and adapted to handling and optimal hygiene. each user. Only a few seconds are needed to define, save or modify your program.

Designed to simplify your daily practice, the Chiropro L unit is infinitely versatile, allowing numerous clinical applications such as implantology, endodontics and dental surgery. Chiropro is a simplified version exclusively dedicated to implantology.

Chiropro L features 7 of the main implant sys- All our physiodispensers are equipped with a

Chiropro L is programmed for the following implant systems: Straumann - Nobel Biocare - Zimmer - Dentsply Implants - Biomet 3i - AstraTech - Thommen Medical

MX-i LED THE BEST MICROMOTOR

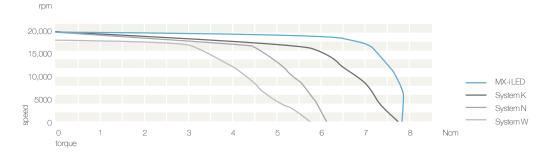

The MX-i LED micromotor is first and foremost a model of stability. It offers great working ease of use at both high and low speeds. The high torque of the MX-i LED makes it one of the most powerful micromotors on the market. Depending on the torque needed, the power is regulated and stabilised by the electronic system. This allows the speed to be maintained with great precision throughout the operation.

Thanks to its lifetime-lubricated ceramic ball bearings, the MX-i LED is sterilisable and yet maintenance-free. This ensures optimal hygiene and start-up, and also offers exceptional reliability and service life.

Other major advantages: the MX-i LED is one of the lightest micromotors ever designed, and provides LED lighting of the contra-angle with the same luminous intensity at both high and low speeds.












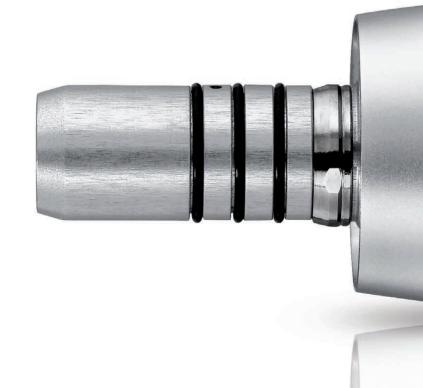



LED lighting



Incomparable lighting quality:
The MX-i LED micromotor incorporates an LED whose luminous intensity can be adjusted according to your sensitivity (10 setting levels).

The LED diffuses a natural white light similar to daylight, with a service life 10 times greater than that of a traditional light bulb.



Bien &







MICRO-SERIES 20:1 L CONTRA-ANGLE ERGONOMICS ABOVE ALL





Bien-Air's innovation guarantees you ever greater ease of use. The new Micro-Series 20:1 L contra-angle is the perfect illustration. It incorporates an internal irrigation system - a world first. The irrigation line is attached at the rear end of the contra-angle, thus freeing you of its cumbersome presence in the hand.

The physiological liquid flows vertically through an orifice in the base of the handpiece. It runs along the bur, thus providing optimum irrigation, especially when fitting implants with surgical guides. To ensure maximum hygiene, Bien-Air has developed a cleaning solution specially dedicated to the internal spray of the contraangle - Aquacare.

the 20:1 L Micro-Series contra-angle is distin- an extremely durable mechanical drive system guished from its competitors by easier access that can withstand screwing forces of up to to the posterior buccal regions. Equipped 70 Ncm, whichever type of tool is used. with an exclusive dual optical glass rod system, the instrument provides uniform lighting of the working area for perfect visibility.

With one of the smallest heads on the market, In addition, the contra-angle is equipped with





TECHNICAL DATA





iCHIROPRO	
Micromotor voltage	1 quick-connect connection/32 Vdc for MX-i LED micromotor
Insulation class	Class I, according to IEC 60601-1
Protection class	IP40
Dim. (LxDxH)/weight	242 x 244 x 102 mm (H with 482 mm bracket)/2.8 kg
Peristaltic pump	Flow rate 30-150 ml/min
Bracket	Stainless steel, height 440 mm
iChiropro ref.	1600784-001
iChiropro set with Micro-Series CA 20:1 L	1700440-001
iChiropro set without Micro-Series CA 20:1 L	1700439-001







CHIROPRO L/CHIROPRO	
Micromotor voltage	1 quick-connect connection/32 Vdc for MX-i LED and MX-i micromotors
Insulation class	Class I, according to IEC 60601-1
Protection class	IP40
Dim. (LxDxH)/weight	$309\times220\times123$ mm (H with 506 mm bracket)/2.7 kg
Peristaltic pump	Flow rate 30-150 ml/min
Irrigation bracket	Stainless steel, height 440 mm
Chiropro L ref.	1600613-001
Programs	8 pre-programmed 10-step sequences 1 endo sequence, 4 programs for dental surgery.
Chiropro L set with Micro-Series CA 20:1 L	1700307-001
Chiropro L set without CA	1700298-001
Chiropro ref.	1600724-001
Programs	8 pre-programmed sequences (10 steps each)
Chiropro set with CA 20:1	1700388-001
Chiropro set without CA	1700387-001

MX-i CABLE	
Туре	Brushless, self-ventilated
Torque	6.8 Ncm
Speed range	100-40,000 rpm
Weight	119 g
Coupling	Compatible with type E coupling as per ISO 3964
Max noise level	55 dBA at 45 cm at 40,000 rpm as per ISO 11498
Cleaning/Sterilisation	Thermal disinfection and sterilisation possible without protection
Ref. MX-i LED micromotor	1600755-001
Light	LED with adjustable intensity, 0-38 klux
Ref. MX-i micromotor	1600825-001
Light	-
Cable ref.	1600606-001



MICRO-SERIES CA 20:1	L/CA 20:1 L/CA 20:1
MICRO-SERIES CA 20:1	L KM/CA 20:1 L KM

Туре	Contra-angle with internal or Kirschner/Meyer irrigation, with light (L) or without light
Transmission ratio	20:1
Speed range	5-2,000 rpm with iChiropro, Chiropro L or Chiropro
Coupling	Compatible with type E coupling as per ISO 3964 (excluding Micro-Series)
Locking mechanism	Push-button bur locking
Bur dimension	ø 2.35 mm / type 1, as per ISO 1797
Cleaning	Thermo Washer Disinfectable
Sterilisation	Steam autoclave up to 135 °C
Micro-Series CA 20:1 L weight	79 g
CA 20:1 L weight	89 g
CA 20:1 weight	89 g
Micro-Series Kirschner/Meyer CA 20:1 L weight	80 g
Kirschner/Meyer CA 20:1 L weight	90 g
Max. torque	70 Ncm
Height of head (incl. 22.5 mm bur)	24.15 mm
Micro-Series CA 20:1 L ref.	1600692-001
CA 20:1 L ref.	1600598-001
CA 20:1 ref.	1600632-001
Micro-Series Kirschner/Meyer CA 20:1 L ref.	1600786-001
Kirschner/Meyer CA 20:1 L ref.	1600785-001









TECHNICAL DATA



PM 1:1 EXTERNAL IRRIGATION	
Туре	Straight handpiece
Transmission ratio	1:1 (direct)
Speed range	100-40,000 rpm*
External irrigation	On detachable sleeve
Detachable sleeves	With or without irrigation tube, supplied with each instrument
Coupling	Type E as per ISO 3964
Locking mechanism	Locking ring
Bur dimension	ø 2.35 mm / type 2, as per ISO 1797
Cleaning	Thermo Washer Disinfectable
Sterilisation	Steam autoclave up to 135 °C
Ref.	1600052-001
Sleeve with irrigation hose	Ref. 1500552-001
Sleeve without irrigation hose	Ref. 1500003-001



PM 1:2 EXTERNAL IRRIGATION	
Туре	Straight handpiece
Transmission ratio	1:2
Speed range	200-80,000 rpm*
External irrigation	On detachable sleeve
Detachable sleeves	With irrigation tube, supplied with each instrument and Bur Guard sterile sleeve
Coupling	Type E as per ISO 3964
Locking mechanism	Locking ring
Bur dimension	ø 2.35 mm / type 2, as per ISO 1797
Cleaning	Thermo Washer Disinfectable
Sterilisation	Steam autoclave up to 135 °C
Ref.	1600436-001
Sleeve with irrigation hose	Ref. 1501318-001
Bur guard sterile sleeve	Ref. 1501317-100

PROTECTIVE FILM FOR IPAD

Set of 10 sterile single-use protective films

Ref. 1501746-010



AQUACARE	SPRAYNET	LUBRIFLUID
Aquacare	Clean with Spraynet®	Lubricate with Lubrifluid®
Cleaning residual physiological liquid	Universal cleaning product for internal and external application	Lubricant for handpieces
Ref. 1600617-006	Ref. 1600036-006	Ref. 1600064-006



IRRIGATION LINES	
Materials	PVC, silicone
Length	2.3 m
External/internal diameter	ø 2.6 mm/ø 1.4 mm
Medical device	Class I
	Sterile, single use
Irrigation lines	Ref. 1500984-010
Kirschner/Meyer irrigation lines	Ref. 1501635-010
Clips	Ref. 1303711-010



KIRSCHNER/MEYER DETACHABLE IRRIGATION SET	
Sterilisation	Steam autoclave up to 135 °C
Kirschner/Meyer detachable irrigation set	Ref. 1501621-010



3-BUTTON FOOT PEDAL	
Description	3 selector buttons, 1 variable speed drive. Compatible with iChiropro, Chiropro L and Chiropro
Functions	Progressive speed, ON/OFF peristaltic pump, program selection, selection of the micromotor's direction of rotation
Protection class	IPX8 as per IEC529
Ref.	1600631-001

