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English

Type

All the benefits of fast connection and easy rotation through 360°, for 3 and 4-hole connection, of air turbines and micromotors, with or without light.

Intended use

Product intended for professional use only. Use in dentistry for prophylaxis, general dentistry and endodontic work.

Technical data and assembly

Connector for 2- or 3-hole attachment **fig. 2**. Connector for 4-hole attachment **fig. 3-4**.

Connector for 4-hole electrified attachment/4VLM **fig. 5**.

Connector for 4-hole electrified attachment/4VLM with air non-return valve **fig. 6**.

Classification

Class IIa, «active medical devices», as per section III «Classification»,

Rule 2 of Annex IX to European Directive 93/42/EEC, namely: «Non-invasive devices intended for channelling liquids or gases for the purpose of administration into the body, which may be connected to an active medical device in Class IIa».

Connections fig. 1

- Screw the tube onto the back of the Unifix.
- To place the turbine on the Unifix, push the turbine as far as the stop. Then check the coupling by pulling on the turbine.
- To remove the turbine, press the buttons of the Unifix as far as the stop and take out the turbine. Proceed to this operation once the turbine has reached a complete stop.

1a All Bien-Air Dental turbines for Unifix connector.

1b Ring for controlling water flow-rate.

1c 4 Bien-Air Dental electrified lines. The Unifix connectors are compatible. You can attach a turbine without light to the connector of a Unifix with light and vice versa.

Polarity **fig. 7**.
Adaptor application **fig. 8**.

Maintenance

Non-sterilisable

Only use maintenance products and components from Bien-Air Dental. The use of other products and components can void the guarantee.

Overhaul

Never disassemble the device. For any modification and repair, we recommend that you contact your regular supplier or Bien-Air Dental directly. Bien-Air Dental asks the user to have its dynamic instruments checked or inspected at least once a year.

Cleaning

Carefully clean all surfaces with a cloth impregnated with **Spraynet**, **fig. 9**.

Disinfection

Disinfect using a clean cloth soaked in a suitable product. Products con-

taining acetone, chlorine and bleaches are not recommended as disinfectants.

Transport

Temperature between -40°C (-40°F) and 70°C (158°F), relative humidity between 10% and 100%, atmospheric pressure 50 kPa to 106 kPa (7.3 to 15.3 psi).

Information

The technical specifications, illustrations and dimensions contained in these instructions are given only as a guide. They may not be the subject of any claim. The manufacturer reserves the right to make technical improvements to its equipment, without amending these instructions. For all additional information, please contact Bien-Air Dental SA at the address indicated on the back cover.

Other precautions for use

The device must be used by a qualified person in accordance with the current legal provisions concerning industrial safety, health and accident prevention measures, and these working instructions. In accordance with these requirements, the operators:

- must only use operating devices that are in perfect working order; in the event of irregular functioning, excessive vibration, abnormal heating or other signs indicating malfunction of the device, the work must be stopped immediately; in this case, contact a repair centre that is approved by Bien-Air Dental;
- must ensure that the device is used only for the purpose for which it is intended, must protect themselves, their patients and third parties from any danger, and must avoid contamination through the use of the product.

The device is intended for medical treatment only; any use other than

that for which this product is intended is unauthorised and may be dangerous. The medical device meets all the current legal requirements.

The device is not authorised for use in an explosive atmosphere (anaesthetic gas).

Never immerse in disinfectant solutions.

Recommendations

It is essential to use dry, purified compressed air in order to ensure the long working life of the device. Maintain the quality of the air and the water by regular maintenance of the compressor and the filtration systems. The use of unfiltered hard water will lead to early blockage of the tubes, connectors and spray cones.

Rest the device on a suitable support to avoid risks of infection for yourself, the patient or third parties.

Guarantee

Terms of guarantee

Bien-Air Dental grants the user a guarantee covering all functional defects, material or production faults. The device is covered by this guarantee for 12 months from the date of invoicing.

In the event of justified claim, Bien-Air Dental or its authorised representative will fulfil the company's obligations under this guarantee by repairing or replacing the product free of charge. Any other claims, of whatever nature, in particular in the form of a claim for damages and interest, are excluded.

Bien-Air Dental shall not be held responsible for damage or injury and the consequences thereof, resulting from:

- excessive wear and tear
- improper use
- non-observance of the instructions for installation, operation and maintenance
- unusual chemical, electrical or

electrolytic influences

- poor connections, whether of the air, water or electricity supply.

The guarantee does not cover flexible "fibre optic" type conductors, or any parts made of synthetic materials.

The guarantee shall become null and void if the damage and its consequences are due to improper manipulation of the product, or modifications to the product carried out by persons not authorised by Bien-Air Dental.

Claims under the terms of the guarantee will be considered only on presentation, together with the product, of the invoice or the consignment note, on which the date of purchase, the product reference and the Serial No. should be clearly indicated.

REF	Legend	List of Bien-Air Dental SA registered trade mark products ®:			
1600363-001	Unifix 4-hole coupling air non-return valve, for instruments with light	Aquilon®	Diamline®	Isolite®	PowerCare®
1600086-001	Unifix 4-hole coupling with spray adjuster for fibre-optic instruments	Bora®	Eolia®	Lubrifluid®	Prestilina®
1600083-001	Unifix 4-hole coupling with spray adjuster	Boralina®	Gyro®	Lubrimed®	Spraynet®
1600082-001	Unifix 4-hole coupling	ChiroPro®	Gyrolina®	MX®	
1600081-001	Unifix 3-hole coupling				
1500010-001	3-hole connector for braided or smooth hoses				
1600089-001	3-hole exit adaptor for standard 4-hole hoses				
1302401-010	Seal 3V				
1300155-010	Seal 4 VL				
1302403-010	Seal 4V				
1300380-010	O-Ring Ø 1.5 mm x Ø 0.75 mm				
1300391-010	O-Ring Ø 7 mm x Ø 1 mm				
1600036-006	Spraynet, cleaning spray 500 ml, box of 6 cans				

a Set supplied

See cover page.

b Optional accessories

See cover page.

In these instructions, “Device” corresponds to the product described in the heading “Type”. For example, turbine, contra-angle, handpiece, micromotor, tube, electronics, connectors, station etc.

Symbols

Manufacturer.

360° rotating coupling.

CE Marking with number of the notified body.

2-hole connection.

3-hole connection.

Attention.

Wear rubber gloves.

Light.

Hose coupling with spray-adjuster.

Movement in the direction indicated.

Movement fully to the stop, in the direction indicated.

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