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English

Type

Multi-purpose hose, diameter 10 mm or 11 mm, 4-hole connector. For turbines, air motors and electric micromotors MC3, with or without light.

Hoses with integral exhaust:

4VLM Ø 11: sheath silicone with electrical socket.

Hoses without integral exhaust:

4VLM Ø 10: sheath silicone with electrical socket.

Intended use

Product intended for professional use only. Use in dentistry for prophylaxis, general dentistry and endodontic work.

Technical data and assembly

Classification

Class IIa, «active medical devices», as per section III «Classification», Rule 2 of Annex IX to European Di-

rective 93/42/EEC, namely: «Non-invasive devices intended for channelling liquids or gases for the purpose of administration into the body, which may be connected to an active medical device in Class IIa».

These hoses are not suitable for pressure higher than 5 bar (500 kPa, 72 psi).

Maximum direct current

2.5 A (5 A 10 s)

Sustained voltage according to

CEI 60601-1

500 Vac/1 min

Weight

4VLM Ø 11

170 g (6 oz)

4VLM Ø 10

142 g (5 oz)

Air delivery at 2.5 bar

> 45 NL/min

Water flow-rate at 2.5 bar

> 200 ml/min

Water-spray at 2.5 bar

> 12 NL/min

Assembly fig. 1

Preserve the initial alignment of the wires in tubes. Place the metal reinforcement ring REF 004.26.26 in the chucking zone **fig. 1a**. Slightly pull the security string **fig. 1b**. For the tubes without string, except coiled hose, select the red or the brown wire and apply the principle of the security string **fig. 1b**. Resistance to traction: 60 N maximum.

Description fig. 1

- 1 Ø 1.5/2.5 mm green: water-spray
- 2 Ø 1.5/2.5 mm white: airspray
- 3 (+) red: motor
- 4 (0V) black: motor
- 5 (+) brown: bulb
- 6 (0V) blue: bulb
- 7 Ø 2.8/4.1 mm white: cooling MC3 or drive turbines
- 8 red: return air
- 9 security string
- 10 sheath silicone

Assembly on turbine fig. 3

Unifix quick-couplings 4-hole instruments for optic and non-optic + tube **with** nut.

Assembly on motor fig. 4

Electric motors MC3 + tube **without** nut.

Accessorie

Intermediate nut **fig. 3**.

Maintenance

Non-sterilisable

Cleaning

Carefully clean all surfaces with a cloth impregnated with Spraynet, **fig. 5**.

Disinfection

Disinfect using a clean cloth soaked in a suitable product. Products containing acetone, chlorine and bleaches are not recommended as disinfectants.

To keep the surface of the hose in good condition, it is advisable to periodically wipe the complete length of it with a cloth dusted with talcum powder.

Never immerse in disinfectant solutions.

Not suitable for ultrasonic bath.

Transport and storage conditions

Temperature between -40°C (-40°F) and 70°C (158°F), relative humidity between 10% and 100%, atmospheric pressure 50 kPa to 106 kPa (7.3 to 15.3 psi).

Information

The technical specifications, illustrations and dimensions contained in these instructions are given only as a guide. They may not be the subject of any claim. The manufacturer reserves the right to make technical improvements to its equipment, without amending these instructions. For all additional information, please contact Bien-Air Dental SA at the address indicated on the back cover.

Other precautions for use

The device must be used by a qual-

fied person in accordance with the current legal provisions concerning industrial safety, health and accident prevention measures, and these working instructions. In accordance with these requirements, the operators:

- must only use operating devices that are in perfect working order; in the event of irregular functioning, excessive vibration, abnormal heating or other signs indicating malfunction of the device, the work must be stopped immediately; in this case, contact a repair centre that is approved by Bien-Air Dental;
- must ensure that the device is used only for the purpose for which it is intended, must protect themselves, their patients and third parties from any danger, and must avoid contamination through the use of the product.

The device is intended for medical treatment only; any use other than that for which this product is intended is unauthorised and may be dangerous. The medical device meets all

the current legal requirements.

The device is not authorised for use in an explosive atmosphere (anaesthetic gas).

Recommendations

It is essential to use dry, purified compressed air in order to ensure the long working life of the device. Maintain the quality of the air and the water by regular maintenance of the compressor and the filtration systems. The use of unfiltered hard water will lead to early blockage of the tubes, connectors and spray cones.

Rest the device on a suitable support to avoid risks of infection for yourself, the patient or third parties.

Only use maintenance products and components from Bien-Air Dental. The use of other products and components can void the guarantee.

Guarantee

Terms of guarantee

Bien-Air Dental grants the operator a guarantee covering all functional defects, material or production faults. The device is covered by this guarantee for 12 months from the date of invoicing.

Bien-Air Dental offers a 24-month guarantee for the glass-bar light conductors.

In the event of justified claim, Bien-Air Dental or its authorised representative will fulfil the company's obligations under this guarantee by repairing or replacing the product free of charge. Any other claims, of whatever nature, in particular in the form of a claim for damages and interest, are excluded.

Bien-Air Dental shall not be held responsible for damage or injury and the consequences thereof, resulting from:

- excessive wear and tear
- improper use

- non-observance of the instructions for installation, operation and maintenance
- unusual chemical, electrical or electrolytic influences
- poor connections, whether of the air, water or electricity supply.

The guarantee does not cover flexible "fibre optic" type conductors, or

any parts made of synthetic materials.

The guarantee shall become null and void if the damage and its consequences are due to improper manipulation of the product, or modifications to the product carried out by persons not authorised by Bien-Air Dental.

Claims under the terms of the guar-

antee will be considered only on presentation, together with the product, of the invoice or the consignment note, on which the date of purchase, the product reference and the Serial No. should be clearly indicated.

REF	Legend	List of Bien-Air Dental SA registered trade mark products ®:			
1600097-001	4VLM Ø 11, with integral exhaust, 4-hole connector with electrical socket. Grey silicone hose	Aquilon®	Diamline®	Isolite®	PowerCare®
1600098-001	4VLM Ø 11, with integral exhaust, 4-hole connector with electrical socket. Black silicone hose	Bora®	Eolia®	Lubrifluid®	Prestilina®
1600102-001	4VLM Ø 10, 4-hole connector with electrical socket, grey silicone hose	Boralina®	Gyro®	Lubrimed®	Spraynet®
1600103-001	4VLM Ø 10, 4-hole connector with electrical socket, black silicone hose	ChiroPro®	Gyrolina®	MX®	
1300876-010	Intermediate nut				
1600036-006	Spraynet, cleaning spray 500 ml, box of 6 cans				

a Set supplied

See cover page.

b Optional accessories

See cover page.

Manufacturer.

CE Marking with number of the notified body.
0123

Wear rubber gloves.

Move in the direction indicated.

Attention.

Move fully to the stop, in the direction indicated.

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