www.bienair.com  
 Other addresses available at  
 dental@bienair.com  
 Tel. +41 (0)32 344 64 64 Fax +41 (0)32 344 64 91  
 Langgasse 60 Case postale 2500 Biel/Bienne 6 Switzerland  

## CA 1:1 CA 30:1

### Irrigation externe


<b>FRA</b>	Mode d'emploi
<b>DEU</b>	Anleitung
<b>ENG</b>	Instruction
<b>ESP</b>	Instrucciones de uso
<b>ITA</b>	Istruzioni
<b>RUS</b>	Инструкция по эксплуатации
<b>JPN</b>	取扱説明書

REF 2100045-0001/2019.12

### a Assortiment livré / Set supplied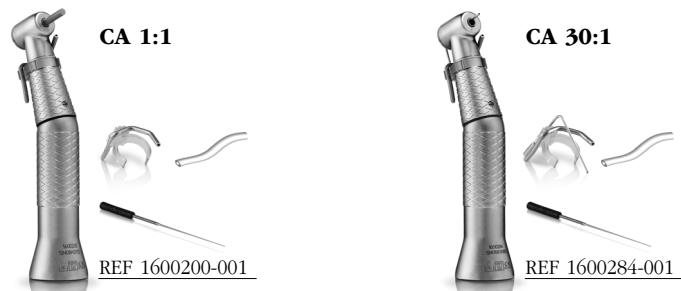

### b Accessoires en option / Optional accessories

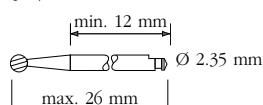
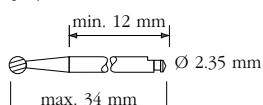
**CA 1:1**Type 1 / ISO 1797  
Code 4-5 / ISO 6360/1**CA 30:1**Type 1 / ISO 1797  
Code 4-6 / ISO 6360/1

fig. 1

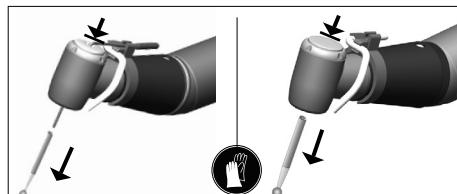
fig. 2

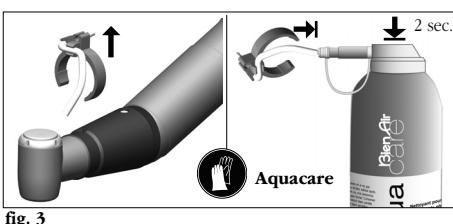
fig. 3

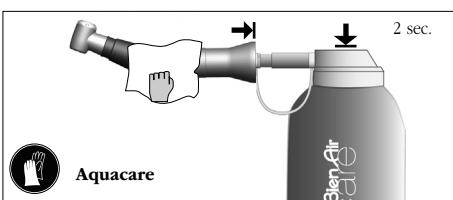
fig. 4

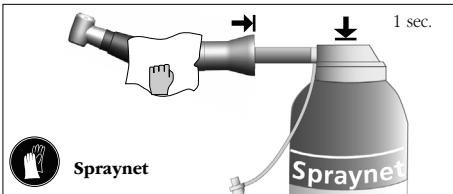
fig. 5

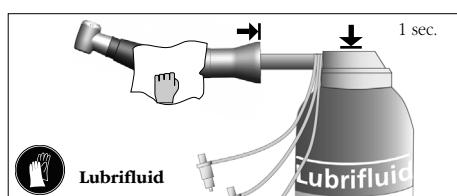
fig. 6

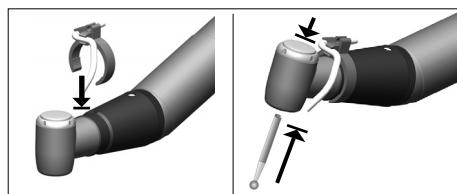
fig. 7

**BienAir<sup>®</sup>**  
**Dental**

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## English

### Type

Contra-angle and reduction contra-angle handpiece without light, with push button, with external irrigation. This contra-angle can be used in endodontics with nickel-titanium NiTi alloy tools.

### Intended use

Product intended for professional use only.

CA 1:1, use in dentistry for implantology, maxillo-facial surgery, general dentistry, prophylaxis and endodontics work.

CA 30:1, use in dentistry for implantology, maxillo-facial surgery and endodontic work.

### Technical data

#### Classification

Class IIa in accordance with European Directive 93/42/EEC concerning medical devices.

#### Type of coupling

The most commonly used coupling

in the world as per ISO Standard 3964.

#### Drive

These contra-angles can be fitted onto an ISO 3964 - compliant coupling motor.

#### Transmission ratio

CA 1:1

Direct drive, blue marker.

CA 30:1

Green marker, reduced by ratio of 30.

#### Motor speed

CA 1:1

max. 40,000 rpm

CA 30:1

max. 40,000 rpm

#### Rotation speed

CA 1:1

max. 40,000 rpm

CA 30:1

max. 1350 rpm

#### Direction of rotation

Clockwise and anticlockwise

### Assembly

#### Important

The device must not be started without a tool inserted into the gripping. Never mount an instrument on a ro-

tating motor.

To ensure your CA micromotor is operating correctly, we recommend a cooling air flow of 7 to 9 standard litres/min.

#### Bur fig. 1.

Diameter of shaft 2,35 mm type 1, in accordance with ISO 1797, max. length CA 1:1, 26 mm code 4-5 and CA 30:1, 34 mm code 4-6, in accordance with ISO 6360/1. In endodontics and implantology, longer instruments can be used under the practitioner's responsibility.

#### Changing the bur

Push-button grip:

Press the push-button **fig. 2** and, at the same time, withdraw the bur. To fit a new bur, press the pushbutton and insert the bur firmly home **fig. 7**.

If the Kirschner/Meyer irrigation system is used, it is recommended to leave the cooling tube in place to change the bur **fig. 2**.

Exert traction on the tool to check that it is correctly in place.

### Maintenance

#### Important

- The instrument is delivered "non sterile".
- Before using for the first time and **within a maximum of 30 minutes** after each treatment, clean, disinfect and lubricate the instrument, then sterilise. Observing this procedure eliminates any blood, saliva or saline solution residues and prevents the transmission system from being blocked.
- Only instruments bearing a **pictogram** can be placed in the washer/disinfectant unit.
- **Do not immerse in an ultrasonic bath.**
- Only use original Bien-Air Dental maintenance products and parts or those recommended by Bien-Air Dental. Using other products or parts may cause operational failure and/or void the guarantee.

### Precautions of use

The standard precautions in particular wearing individual protection

equipment (gloves, glasses ...), must be complied with by healthcare personnel working with contaminated or potentially contaminated medical instruments. Pointed and sharp instruments should be handled with great caution.

Check that the steriliser and the water that is used are clean. After each sterilisation cycle, remove the device from the sterilising apparatus immediately, in order to reduce the risk of corrosion.

### Bur chuck mechanism (cutting tool)

- Carry out cleaning – disinfection – sterilisation without bur in the chuck mechanism.

### Suitable detergent For automatic cleaning-disinfection:

- Slightly alkaline enzymatic detergent recommended for cleaning in a washer-disinfector for dental or surgical instruments.

### For manual cleaning-disinfection:

- Detergent or detergent-disinfect-

tant (pH 6-9.5) recommended for cleaning-disinfection of dental or surgical instruments. Quaternary ammonium- and/or enzyme-based surfactants.

- **Do not use** detergents that are corrosive or contain chlorine, acetone aldehydes or bleaches.
- Do not soak in physiological liquid (NaCl).

#### ⚠ Important

In the event of prolonged disuse, the instrument must be stored in a dry environment. Lubricate and sterilise the instrument before reuse.

After cleaning and disinfecting/ sterilising the instrument and before using it, operate at moderate speed, with a bur in the chuck, for 10 to 15 seconds to circulate lubricant and remove excess.

### O Pre-disinfection

#### Preparation

- Disconnect the attachment from the drive motor and remove the bur **fig. 2**.
- Wash out the physiological liquid from the duct by spraying

(2 secs.) in the tube at the rear with **Aquacare fig. 3**.

- Carry out a manual pre-clean or a bath pre-clean.

#### Manual pre-cleaning

Use **Aquacare** cleaning product.

- Spray for 2 seconds on the inside and outside of the attachment remove of residues and any remaining physiological liquid **fig. 4**.

- Carefully clean the surfaces.
- Leave to drip-dry and wipe the outside surfaces with absorbent paper or immediately proceed with step no 2.

or

#### Pre-cleaning by bath

⚠ Warning: the preliminary bath cleaning **should be followed** by automatic cleaning-disinfection.

- Totally submerge the instrument in a cleaning bath containing the suitable detergent following the manufacturer's instructions (duration, concentration, temperature, renewal...).
- On leaving the bath, rinse the

inside and outside of the device in cold running water (<35°C/ <95°F) for about 30 seconds.

- Leave to drip-dry and wipe the outside surfaces with absorbent paper or immediately proceed with automatic cleaning-disinfection.

### 1 Cleaning-disinfection

#### Manual cleaning-disinfection

##### Maintenance and sterilisation

Soak the instrument in a tray containing the suitable detergent and, using a soft, clean and disinfected brush, clean the exterior of the device following the detergent manufacturer's instructions (duration, concentration, temperature, renewal...).

##### Rinsing

Rinse the inside and outside of the device with cold running water (<35°C/ <95°F) for about 30 sec.

##### Drying

Spray the outside and inside of the device with **Spraynet fig. 5** to remove the rinsing water immediately in order to prevent damage to and jamming of internal parts. If re-

quired, wipe with a sterile non-woven wipe.

**or**

### **Automatic cleaning-disinfection**

**⚠ Note:** only for instruments engraved with ■

#### Cleaner - disinfectant

Use a validated cleaner-disinfectant with the adequate detergent.

#### Washing cycle

Select the specified washing cycle for these instruments which should be compatible with the detergent manufacturer's instructions.

### **Recommended temperatures for the attachments:**

#### Pre-wash

Water: cold to maximum 35°C (95°F) – Duration: 2 mins.

#### Wash

Water: 50 to 65°C (122 to 149°F) Duration 5 mins.

#### Rinsing

Water: cold to maximum 35°C (95°F) – Duration: 2 mins.

### Thermal disinfection

Water: 80 to 97°C (176 to 206.6°F)  
Duration 5 mins.

### Drying

Air: 65 to 75°C/(149 to 167°F)  
Duration 25 mins.

**⚠ Note:** Never rinse the devices to cool them.

### **2 Lubrication, checks and conditioning process**

#### **Verifying cleanliness**

Visually check the cleanliness of the instrument. If required, re-clean with a soft brush.

#### **Lubrication**

Use **Lubrifluid** spray.

- Place the instrument in a cloth to collect all surplus lubricant.
- Insert the spray tip in the back of the instrument handle **fig. 6**.
- Operate the spray for 1 sec.
- Reinstall the sleeve or the clipped tube **fig. 7**.

#### **Clamp operation inspection**

- Insert a bur in the chuck mecha-

nism of the instrument **fig. 7**.

- Check that the bur remains in position when traction is applied.
- Remove the bur.

### **Conditioning process**

Wrap the instrument in approved packaging for steam sterilisation.

### **3 Sterilisation**

#### **⚠ Important**

**The quality of the sterilisation depends very much on the cleanliness of the device.** Only perfectly clean devices may be sterilised.

**Do not use a sterilisation procedure other than the one described below.**

**Procedure:** With steam after fractioned initial vacuum phase, Class B cycle acc. to EN13060. The procedure has been validated according to ISO 17664. Nominal temperature: 135°C (273°F). Duration: 3 or 18 min., according to the national guidelines in force.

#### **Overhaul**

Never disassemble the device. For

any modification and repair, we recommend that you contact your regular supplier or Bien-Air Dental directly. Bien-Air Dental asks the user to have its dynamic instruments checked or inspected at least once a year.

### **Transport**

Temperature between -40°C (-40°F) and 70°C (158°F), relative humidity between 10% and 100%, atmospheric pressure 50 kPa to 106 kPa (7.3 to 15.3 psi).

### **Information**

The technical specifications, illustrations and dimensions contained in these instructions are given only as a guide. They may not be the subject of any claim. The manufacturer reserves the right to make technical improvements to its equipment, without amending these instructions. For all additional information, please contact Bien-Air Dental SA at the address indicated on the back cover.

## **Other precautions for use**

The device must be used by a qualified person in accordance with the current legal provisions concerning industrial safety, health and accident prevention measures, and these working instructions. In accordance with these requirements, the operators:

- must only use operating devices that are in perfect working order; in the event of irregular functioning, excessive vibration, abnormal heating or other signs indicating malfunction of the device, the work must be stopped immediately; in this case, contact a repair centre that is approved by Bien-Air Dental;
- must ensure that the device is used only for the purpose for which it is intended, must protect themselves, their patients and third parties from any danger, and must avoid contamination through the use of the product.

The device is intended for medical treatment only; any use other than that for which this product is intend-

ed is unauthorised and may be dangerous. The medical device meets all the current legal requirements.

Excess material from products used for maintenance (lubricants, cleaning products and disinfectants) originating from CA or PM may penetrate into the electric brush motor and interfere with its functioning. It is essential to follow the maintenance instructions accompanying each product. Never lubricate the electric brush motor.

### **Recommendations**

It is essential to use dry, purified compressed air in order to ensure the long working life of the device. Maintain the quality of the air and the water by regular maintenance of the compressor and the filtration systems. The use of unfiltered hard water will lead to early blockage of the tubes, connectors and spray cones.

The device must not be used in the presence of open lesions, injury to the soft tissues or recent extractions. The exhaust air could propel infect-

ed material into the wounds and cause infections and risk of embolism.

Adhere to the Instructions for Use, in accordance with the instructions of the bur manufacturer. Never use a bur with an incorrect diameter, as there is a risk of this becoming detached during the treatment and causing injury to yourself, to your patient and to third parties.

Rest the device on a suitable support to avoid risks of infection for yourself, the patient or third parties.

## **Guarantee**

### **Terms of guarantee**

Bien-Air Dental grants the user a guarantee covering all functional defects, material or production faults.

The device is covered by this guarantee for 12 months from the date of invoicing.

In the event of justified claim, Bien-Air Dental or its authorised represen-

tative will fulfil the company's obligations under this guarantee by repairing or replacing the product free of charge.

Any other claims, of whatever nature, in particular in the form of a claim for damages and interest, are excluded.

Bien-Air Dental shall not be held responsible for damage or injury and the consequences thereof, resulting from:

- excessive wear and tear
- improper use
- non-observance of the instructions for installation, operation and maintenance
- unusual chemical, electrical or electrolytic influences
- poor connections, whether of the air, water or electricity supply.

The guarantee does not cover flexible "fibre optic" type conductors, or any parts made of synthetic materials.

The guarantee shall become null and void if the damage and its consequences are due to improper manipulation of the product, or modifi-

cations to the product carried out by persons not authorised by Bien-Air Dental.

Claims under the terms of the guarantee will be considered only on presentation, together with the product,

of the invoice or the consignment note, on which the date of purchase, the product reference and the Serial No. should be clearly indicated.

## REF Legend

1600200-001 Contra-angle CA 1:1 direct ratio. Push-button grip.  
Removable external irrigation. Including irrigation set  
REF 1500431-010

1600284-001 Contra-angle CA 30:1, 30x reducer. Push-button grip,  
removable external irrigation and Kirschner/Meyer type  
detachable irrigation system. Including irrigation set  
REF 1500467-010

1500431-010 Removable irrigation set for CA 1:1 comprising 10 rings,  
10 side tubes

1500467-010 Detachable irrigation set for CA 30:1 comprising 10 rings,  
10 side tubes and 10 Kirschner/Meyer type tubes

031.16.42-010 Flexible spray hose, dia. 1.2 / 2.5 x 77

1600617-006 Aquacare, cleaning spray 500 ml, box of 6 cans

1600036-006 Spraynet, cleaning spray 500 ml, box of 6 cans

1600064-006 Lubrifluid, lubricant 500 ml, box of 6 cans

## List of Bien-Air Dental SA registered trade mark products ®:

Aquilon® ChiroPro® Gyrolina® Lubrimed® Prestilina®  
Bora® Eolia® Isolite® MX® Spraynet®  
Boralina® Gyro® Lubrifluid® PowerCare®

In these instructions, "Device" corresponds to the product described in the heading "Type". For example, turbine, contra-angle, handpiece, micromotor, tube, electronics, connectors, station etc.

## Symbols

Manufacturer.

CE Marking with number  
of the notified body.

Wear rubber gloves.

Move in the direc-  
tion indicated.

Thermo washer  
disinfectable.

Attention.

Move fully to the  
stop, in the direction  
indicated.

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