www.bienair.com

Other addresses available at

dental@bienair.com

Länggasse 60 Case postale 2500 Bienne 6 Switzerland
Tel. +41 (0)32 344 64 64 Fax +41 (0)32 344 64 91


 Bien-Air Dental SA

PML 1121

FRA Mode d'emploi
DEU Anleitung
ENG Instruction
ESP Mantenimiento
ITA Istruzioni
RUS Русский

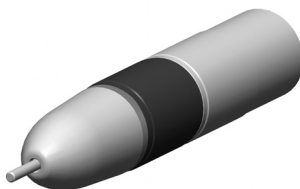REF 2100098-0001/2019.10

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English

Type

Dental straight handpiece, with clamp and reinforced bearings.

Planned use

Use in laboratory work.

Technical data and assembly

Type of coupling

The most commonly used coupling in the world as per ISO Standard 3964 without internal spray, without light.

Coupling

The PML 1121 can be coupled to a micromotor of the ISO Standard 3964 type.

Transmission ratio

Blue instrument sleeve direct ratio

Bur

Diameter of shaft 2,35 mm type 2, in accordance with ISO 1797, max. length 44,5 mm code 4, in accordance with ISO 6360/1, fig 1.

Changing the bur

Turn the ring completely in the direction of arrow:

A: unlocking **fig. 2** **B:** locking **fig. 3**

The references (triangles) are aligned in the closed position.

Important

The device must not be started without a tool inserted into the chuck.

Never mount an instrument on a rotating motor.

Never activate the clamping/unclamping ring while the instrument is rotating, as there is a risk of damaging its mechanism or destroying the motor.

Maintenance

Maintenance animations at www.bienair.com

Separately clean, lubricate and sterilise the device before the application.

Only use maintenance products and/or components from Bien-Air. The use of other products and components can

lead to annulment of the guarantee.

The device is supplied in «non-sterile» form. Before use, please follow the instructions under this heading.

Check that the steriliser and the water that is used are clean. After each sterilisation cycle, remove the device from the sterilising apparatus immediately, in order to reduce the risk of corrosion.

Overhaul

Never disassemble the device. For all modifications and repairs, we recommend that you contact your regular supplier or Bien-Air direct. Bien-Air asks the user to have its dynamic instruments checked or inspected at least once a year thereafter.

Cleaning and disinfection

Use cleaner Spraynet, Bien-Air **fig. 4**. Spray the inside and outside of the equipment to remove the residues. Remove the bur before cleaning and lubricating.

Disinfect with a clean damp cloth. Products containing acetone, chlorine and bleach are not recommended as

disinfectants.

Never immerse in disinfectant solutions.

Not suitable for ultrasonic bath.

Before sterilisation

Remove residue, carefully clean the device.

Lubrication

Lubrication before each sterilisation, and at least twice a day. Only use the spray Lubrifluid.

- Remove tool from instrument gripper before lubrication
- Place the instrument in a cloth so as to catch the debris and foreign matter expelled by the atomizer jet.
- Remove the protective cap and insert the nozzle into the back of the instrument sleeve.
- Spray for about 1 second **fig. 5**.

Sterilisation

Process: Steam autoclave up to 135°C (275°F). The required time for sterilisation of our products must comply with the prevailing local regulations for this type of instrument.

Important

After cleaning and disinfecting/sterilizing the instrument and before using it, operate at moderate speed, with a bur in the chuck, for 10 to 15 seconds to distribute, and remove, the excess lubricant.

Environment

Temperature between -40°C (-40°F) and 70°C (158°F), relative humidity between 10% and 100%, atmospheric pressure 50 kPa to 106 kPa (7,3 to 15,3 psi).

Application

Product intended for professional use only.

Information

The technical specifications, illustrations and dimensions contained in these instructions are given only as a guide. They may not be the subject of any claim. The manufacturer reserves the right to make technical improvements to its equipment, without amending these instructions.

The English original of this text is the reference and conforms to the legal standards. For all additional information, please contact Bien-Air at the address indicated on the reverse.

Generalities

The device must be used by a competent person in accordance with the current legal provisions concerning industrial safety, health and accident prevention regulations, and these working instructions. In accordance with these requirements, the operator:

- must only use working devices that are in perfect working order; in the event of irregular functioning, excessive vibration, abnormal heating or other signs indicating malfunction of the device, the work must be stopped immediately; in this case, contact a repair centre that is approved by Bien-Air;
- must ensure that the device is used only for the purpose for which it is intended, must protect himself and third parties from any danger, and must avoid contamination through the use of the product.

Any use other than that for which this product is intended is not allowed and may prove to be dangerous. The device meets all the current legal requirements.

Excess material from products used for maintenance (lubricants, cleaning products and disinfectants) originating from CA or PM may penetrate into the electric brush motor and interfere with its functioning. It is essential to follow the maintenance instructions accompanying each product.

Recommendations

It is essential to use dry, purified compressed air in order to ensure the long working life of the device. Maintain the quality of the air and the water by regular maintenance of the compressor and the filtration systems. The use of unfiltered hard water will lead to early blockage of the tubes, connectors and spray cones.

Adhere to the Instructions for Use, in accordance with the instructions of the tool manufacturer. Never use a tool with an incorrect rod, as there is a risk

of this becoming detached during the process and causing injury to you yourself and to third parties.

Rest the device on a suitable support in order to eliminate the risk of injury to you yourself and to third parties.

Guarantee

Terms of guarantee

Bien-Air grants the operator a warranty covering all functional defects, material or production faults. The device is covered by this warranty for 12 months from the date of invoicing.

Bien-Air also offers a 24-month guarantee for the glass-bar light conductors. In the event of justified claims, Bien-Air or its authorised representative will fulfil the company's obligations under this guarantee by repairing or replacing the product free of charge. Any other claims, of whatever nature, in particular in the form of a claim for damages and interest, are excluded.

Bien-Air shall not be held responsible for damage or injury and the consequences thereof, resulting from:

- excessive wear and tear
- improper handling
- non-observance of the instructions for installation, operation and maintenance
- unusual chemical, electrical or electrolytic influences
- poor connections, whether of the air water or electricity supply.

The guarantee does not cover flexible «fibre optic» type conductors, or any parts made of synthetic materials.

The guarantee shall become null and void if the damage and its consequences are due to improper manipulation of the product, or modifications to the product carried out by persons not authorised by Bien-Air.

Claims under the terms of the guarantee will be considered only on presentation, together with the product, of the invoice or the consignment note, on which the date of purchase, the product reference and the Serial No. should be clearly indicated.

REF Legend

1600156-001	Laboratory handpiece PML 1121, 1:1 ratio, extra strong bur-locking mechanism
1600064-006	Lubrfluid, lubricant 500 ml, box of 6 cans
1600036-006	Spraynet, cleaning spray 500 ml, box of 6 cans

a

Accessories in option (last page)

b

Set supplied (last page)

List of registered trade mark products ®:

Aquilon®	Gyro®	Lubrfluid®	Spraynet®	Osseodoc®
Bora®	Gyrolina®	Lubrimed®	MX®	Osseopro®
Boralina®	Isolite®	Prestilina®		

In these instructions, «Device» corresponds to the product described in the heading «Type». For example, turbine, contra-angle, handpiece, motor, tube, electronics, connectors, station etc.

Symbols

Wear rubber gloves

CE conformity Marking

Movement in the direction indicated

Movement fully to the stop, in the direction indicated

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