Other addresses available at
www.bienair.com

Bien-Air Dental SA
Länggasse 60 Case postale 2500 Bienne 6 Switzerland
Tel. +41 (0)32 344 64 64 Fax +41 (0)32 344 64 91
dental@bienair.com

Hoses MCX & MCX c.l. MCX-400 & MCX-400 c.l. B-MCX

Ne pas stériliser / Do not sterilise

ENG	Instruction
FRA	Mode d'emploi
DEU	Anleitung
ESP	Instrucciones de uso
ITA	Istruzioni
RUS	Инструкция по эксплуатации
JPN	取扱説明書

REF 2100230-0003/2019.10

Assortiment livré / Set supplied

MCX

REF 1600756-001
REF 1600756-0

MCX c.l.

REF 1600854-001
REF 1600854-0

B-MCX

REF 1600824-001
REF 1600824-0


MCX-400

REF 1601081-001
REF 1601081-0

Accessoires en option / Optional accessories

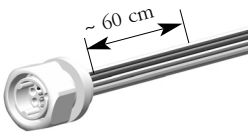REF 1600753-001

REF 1300367-010

REF 1300368-010

REF 1600036-006

MCX-400 c.l.

REF 1601096-001
REF 1601096-0

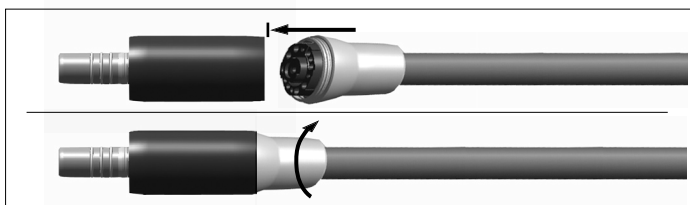
fig. 1

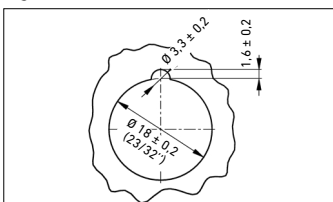
fig. 2

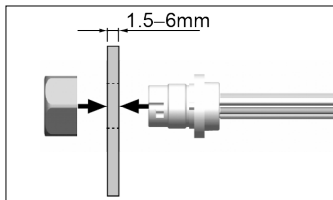
fig. 3

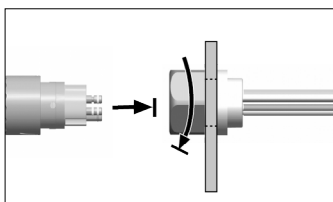
fig. 4

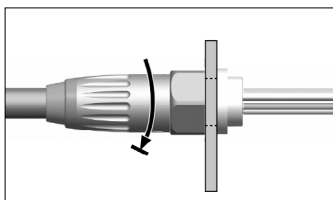
fig. 5

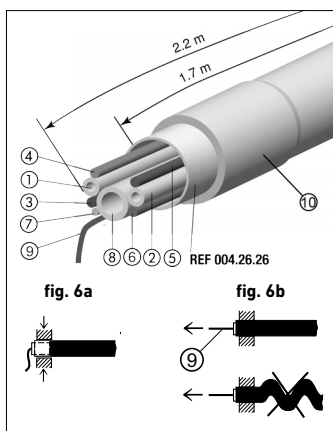
fig. 6

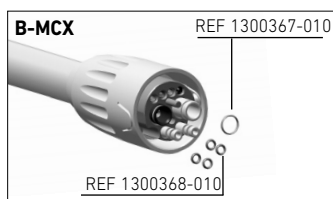
fig. 7

fig. 8

BienAir[®]
Dental

This product may be covered by one or more of the following patents:

EP Europe: 745358 / 688539 / 948294 / 1145688 / 1563800 / 1563801 / 1675523 / 1753360 **DE Germany:** 29616023.7

DK Denmark: 9600315 **FR France:** 2722972 **CH Switzerland:** 693922 **CN China:** 100528099 / 100522100 / 100522099 / 100553584

JP Japan: 3892485 / 4298933 / 7000419 **US United-States:** 5453008 / 6033220 / 6319003 / 7214060 / 7448870

RU Russia: 2361540 / 2361541 / 2372046

REF 1600756-001 HOSE MCX / REF 1600824-001 HOSE B-MCX / REF 1601081-001 HOSE MCX-400 / REF 1601096-001 HOSE MCX-400 C.L.

Garantie · Garantie
Guarantee · Garantía
Garanzia · Гарантия
保証

12

Mois · Monate
Months · Meses
Mesi · Месяцев
ヶ月

English	Intended use	2
	Technical data and assembly	2
	Maintenance	2
	Other precautions for use	3
	Guarantee	3
	REF / Legend	4

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English

In these instructions, "Device" corresponds to the product described in the heading "Identification". For example, turbine, contra-angle, handpiece, micromotor, tube, electronics, connectors, station etc.

Identification

Grey silicone hoses, for MCX electric micromotor with light or without light

MCX

with screw connector

MCX c.l.

with screw connector, long cap

B-MCX

with bayonet connection to unit.

MCX-400

with screw connector, rotating

MCX-400 c.l.

with screw connector, rotating, long cap

Intended use

Product intended for professional use only. Use in dentistry for prophylaxis, general dentistry and endodontic work. Any use other than that for which this device is intended is prohibited and may prove dangerous.

Technical data

These hoses are not suitable for pressure higher than 5 bar (500 kPa, 72 psi).

Classification

Class IIa in accordance with European Directive 93/42/EEC concerning medical devices. This medical device is in compliance with the legislation in force.

Standard length

1.70 m

Maximum direct current

2.5 A (5 A/10 s)

Sustained voltage according to IEC 60601-1

500 Vac/1 min

Flow rate of the air-intake tube at 2.5 bar (36 psi)

62 standard litres/min

Flow rate of the air-spray tube at 2.5 bar (36 psi)

7.6 standard litres/min

Flow rate of the water-spray tube at 2 bar (29 psi)

170 ml/min

Information

The technical specifications, illustrations and dimensions contained in these instructions are given only as a guide. They may not be the subject of any claim. The manufacturer reserves the right to make technical improvements to its equipment, without amending these instructions. For all additional information, please contact Bien-Air Dental SA at the address indicated on the back cover.

Assembly

Assembly on motor fig. 1

Electric micromotor MCX.

Installing the socket in the unit fig. 2-5

Drilling for the wall $\varnothing 18 \text{ mm} \pm 0.2$ (23/32")
Machine a countersink for the catch **fig. 2**

Tighten nut 150 Ncm maximum
Minimum wall thickness 1.5 mm
Maximum wall thickness 6 mm

Assembly MCX fig. 6

Preserve the initial alignment of the wires and tubes. Place the metal reinforcement ring REF 004.26.26 in the chucking zone **fig. 6a**.

Attach the securing cord to avoid any strain on the wires and tubes **fig. 6b**.

Resistance to traction: 60 N maximum.

Description fig. 6

- 1 $\varnothing 1.5/2.5$ mm green: waterspray
- 2 $\varnothing 1.5/2.5$ mm white: airspray
- 3 Phase A motor, blue
- 4 Phase B motor, red
- 5 Phase C motor, black
- 6 + light, brown
- 7 - light, orange
- 8 $\varnothing 2.8/4.1$ mm white: cooling motors
- 9 securing cord
- 10 silicone sheath

Accessories

For bayonet connectors, **fig. 7**: O-ring connections REF 1300367-010 and 1300368-010.

Quick connector on unit side only for $\varnothing 18$ mm (23/32") hole, with rotation prevention notch. Standard length 60 cm, REF 1600753-001.

Disposal

This device must be recycled. Electrical and electronic equipment may contain dangerous substances which constitute health and environmental hazards. The user must return the device to its dealer or establish direct contact with an approved body for treatment and recovery of this type of equipment (European Directive 2002/96/EC).

Maintenance

Non-sterilisable

Cleaning

Use **Spraynet** from Bien-Air Dental **fig. 8**. Spray the outside of the equipment and carefully clean the surface.

Disinfection

Disinfect using a clean cloth soaked in a suitable product. Products containing acetone, chlorine and bleaches are not recommended as disinfectants. To keep the surface of the hose in good condition, it is advisable to periodically wipe the complete length of it with a cloth dusted with talcum powder.

Never immerse in disinfectant solutions.

Not suitable for ultrasonic bath.

Only use maintenance products and components from Bien-Air Dental. The use of other products and components can void the guarantee.

Transport and storage conditions

Temperature between -40°C (-40°F) and 70°C (158°F), relative humidity between 10% and 100%, atmospheric pressure 50 kPa to 106 kPa (7.3 to 15.3 psi).

Other precautions for use

The device must be used by a qualified person in accordance with the current legal provisions concerning industrial safety, health and accident prevention measures, and these working instructions. In accordance with these requirements, the operators:

- must only use operating devices that are in perfect working order; in the event of irregular functioning, excessive vibration, abnormal heating or other signs indicating malfunction of the device, the work must be stopped immediately; in this case, contact a repair centre

approved by Bien-Air Dental;

- must ensure that the device is used only for the purpose for which it is intended, must protect themselves, their patients and third parties from any danger, and must avoid contamination through the use of the product.

Rest the device on a suitable support to avoid risks of infection for yourself, the patient or third parties.

The device is not authorised for use in an explosive atmosphere (anaesthetic gas).

Recommendations

It is essential to use dry, purified compressed air in order to ensure the long working life of the device. Maintain the quality of the air and the water by regular maintenance of the compressor and the filtration systems. The use of unfiltered hard water will lead to early blockage of the tubes, connectors and spray cones.

Guarantee

Terms of guarantee

Bien-Air Dental grants the operator a guarantee covering all functional defects, material or production faults.

The device is covered by this guarantee for 12 months from the date of invoicing.

In the event of justified claim, Bien-Air Dental or its authorised representative will fulfil the company's obligations under this guarantee by repairing or replacing the product free of charge.

Any other claims, of whatever nature, in particular in the form of a claim for damages and interest, are excluded.

Bien-Air Dental shall not be held responsible for damage or injury and the consequences thereof, resulting from:

- excessive wear and tear
- improper use
- non-observance of the instructions for installation, operation and maintenance
- unusual chemical, electrical or electrolytic influences
- poor connections, whether of the air, water or electricity supply.

The guarantee does not cover flexible "fibre optic" type conductors, or any parts made of synthetic materials.

The guarantee shall become null and void if the damage and its consequences are due to improper manipulation of the product, or modifications to the product carried out by persons not authorised by Bien-Air Dental.

Claims under the terms of the guarantee will be considered only on presentation, together with the product, of the invoice or the consignment note, on which the date of purchase, the product reference and the Serial No. should be clearly indicated.

REF	Legend
1600756-001	MCX hose, grey silicone
1600756-0	MCX hose, special length
1600854-001	MCX c.l. hose, long cap, grey silicone
1600854-0	MCX c.l. hose, long cap, special length
1600824-001	B-MCX hose, grey silicone, bayonet connection to unit
1600824-0	B-MCX, hose, special length
1601081-001	MCX-400 hose, rotating, grey silicone
1601081-0	MCX-400 hose, rotating, special length
1601096-001	MCX-400 c.l. hose, rotating, long cap, grey silicone
1601096-0	MCX-400 c.l. hose, rotating, long cap, special length
1600753-001	Quick connector on unit side only for Ø 18 mm hole, with rotation prevention notch. Standard length 60 cm
1300367-010	O-ring Ø 3.2 x 0.6
1300368-010	O-ring Ø 1.5 x 0.75
1600036-006	Spraynet, cleaning spray 500 ml, box of 6 cans

Sets available

see cover page.

Optional accessories

see cover page.

Symbols

Manufacturer.

Wear rubber gloves.

Attention.

CE Marking with number of the notified body.

Move in the direction indicated.

Move fully to the stop, in the direction indicated.

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