

Quick start guide to activate WaveOne® Gold



1

Install and run the **iOptima application** that you can find on the app store



Connect your iPad Mini / iPod to the docking station. Click on the following buttons and take note of the displayed ID code.



2

Go to the webpage www.ioptima-waveone.com

1. Enter your personal information.
2. Enter the **license code** that you can find in the CA Endo Bien-Air box.
3. Enter the ID code that you previously found in the iOptima app (see step 1).


Name	First name
<input type="text"/>	<input type="text"/>
Dental office	<input type="text"/>
Enter your dental office	<input type="text"/>
Email address	<input type="text"/>
Enter your email address	<input type="text"/>



ID code
XXXX-XXXX

3

4. Click on the button « **Generate the activation code** »
5. Your activation code gets now displayed

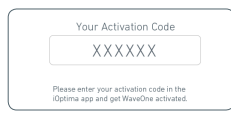

4

Run the **iOptima application**

Click on the following buttons and enter the activation code.



Congratulations!
You can now fully enjoy the

WaveOne® Gold
systems